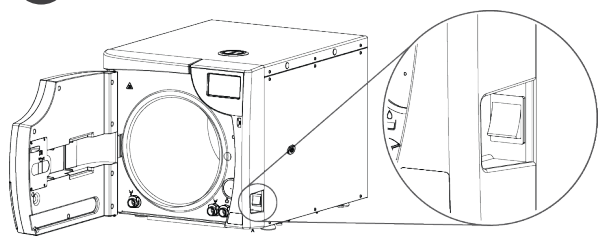
SciCan BRAVO G4 Quick Reference Guide

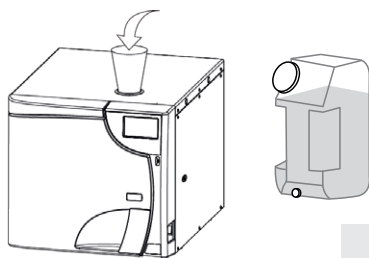
Operator's Manual (located on the USB key) should be consulted before using this autoclave.

BASIC OPERATION

1 Power unit ON.


2 Fill with demineralized/distilled water.


NEVER USE TAP WATER

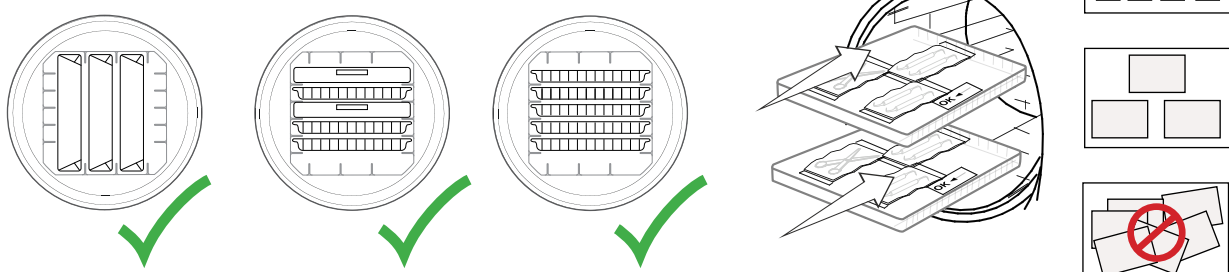3 Load configurations.

4 Close unit door.

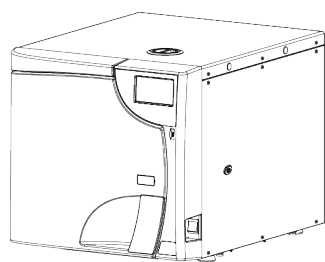
5 Run cycle.

Select a cycle.


Start a cycle.

End of cycle.

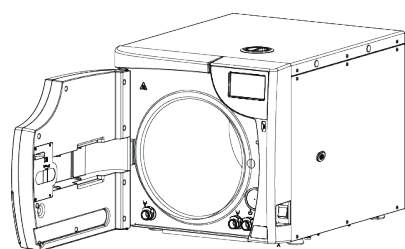
6 Open door.


7 Unload instruments.

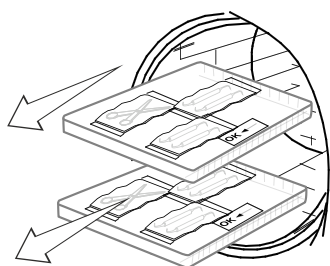
8 If required, drain manually.

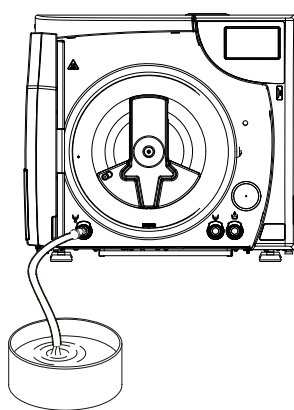
TOUCHSCREEN

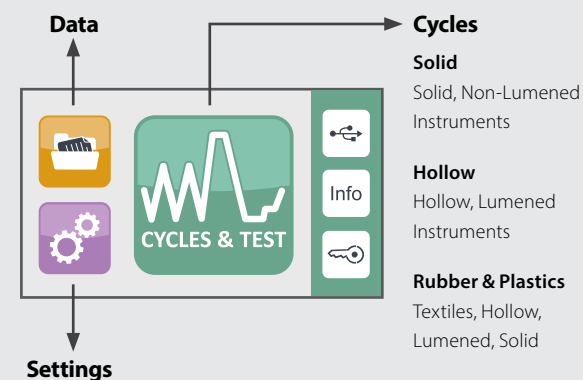
VENTILATION

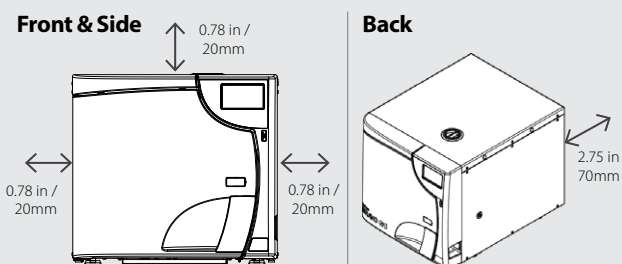FILLING OPTIONS

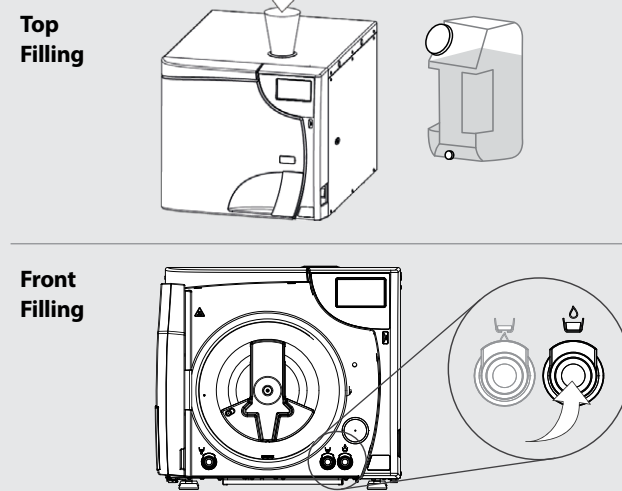DRAINING OPTIONS

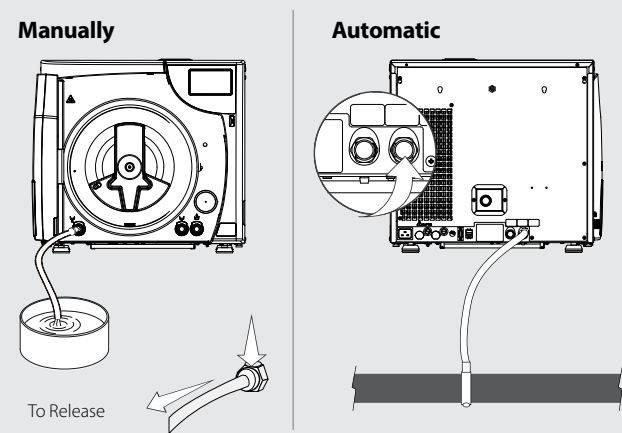MAINTENANCE SCHEDULE

Daily	Clean the door gasket and internal part of the door with a clean wet cloth. Check for wear and replace if required.
	Completely empty the waste tank before refilling the water reservoir.
Weekly	Clean the chamber and appropriate accessories with a wet cloth. If required, use a neutral detergent (without alkalines and bleaches). Carefully rinse with demineralized/distilled water and remove any residue.
	Inspect and clean chamber anti-dust filter . Replace if necessary.
	Ensure area around BRAVO G4 is dust free.
Monthly	Drain & clean the internal water reservoir as per the instructions outlined in the Operator's Manual.
	If applicable, clean the external water tank used for automatic filling.

	Check the bacteriological filter and replace as required.
	Lubricate the door lock . Using a clean cloth, remove any residues and lubricate the inside of the bushing with a film of the silicone-based grease provided.
Every 3 - 6 Months	Inspect and clean the chamber drain filter . Replace if required.
Every Year or 1,000 Cycles	Replace the door gasket .


Spare Parts and Accessories	
97400258	Door Gasket 17L/22L
97290160	Bacteriological Filter
97290210	Chamber Drain Filter
99951785	Chamber Rack 17L
99951786	Chamber Rack 22L
99950394	Pouch Rack
99951317	Tray 17L
99951318	Tray 22L
97467066	Tray Extractor

IMPORTANT!

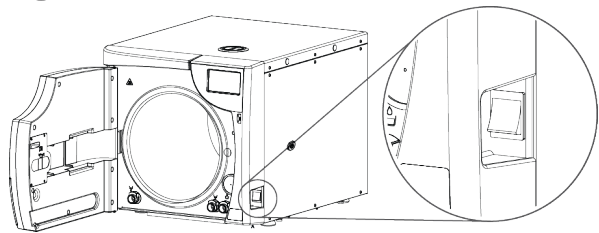Users of this device must be sure to select the most appropriate sterilization cycle according to the recommendations of the leading infection control authorities and local regulatory guidelines / recommendations.

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